

## DO NOT PRINT LEGEND

**Helen  
of Troy**

**Creative Department  
Artwork Specifications**

**Braun**

Thermometers

**BNT050WE**

Artwork Part #: **A006942R2**

Dieline Part #: **N/A**

Subject: **Quick Start Guide**

Region: **EMEA**

Flat Size: **MM: W266.7 x H101.6**

**IN: W10.5 x H4**

Folded Size: **MM: W38.1 x H101.6**

**IN: W1.5 x H4**

Scale: **1/1**

Material: **120g SBS (art paper)**

# of Pages: **1 page front and back**

Revision: **4**

Date: **30JUL24**

Release Date: **12AUG24**

Re-Release Date: **00MON00**

### Colors

Cyan  
100%

Magenta  
100%

Yellow  
100%

Black (K)  
100%



Dielines do not print


Coating: **Coating**

Special Instructions:


Quality Requirement of Artwork  
**Clarification Process of Artwork Printing: Meet  
Eng-QS-06&02**

**Helen of Troy Creative Services**  
Marlborough, MA 01752 USA  
Questions? Contact your representative


# 1 Taking a measurement

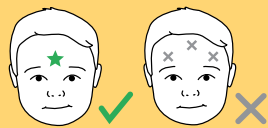


Aim between eyebrows.

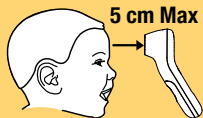

Maximum distance.

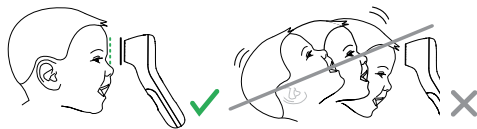


Keep head still.

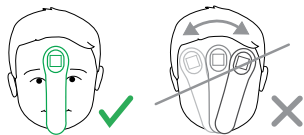


Keep thermometer in correct position.



# 2 Temperature taking tips

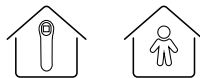

Before measurement: Keep thermometer in room temperature for 10 minutes and individual indoors for 30 minutes.

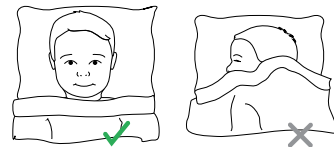

# BRAUN

Sensian™ 3


## Los geht's!

## C'est parti!


## Let's get started!

BNT050


# 1 Durchführen einer Messung Prendre la température

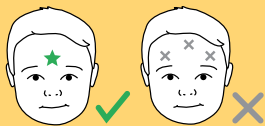


Richten Sie das Thermometer  
zwischen die Augenbrauen.  
Visez entre les sourcils.

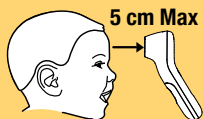

Maximale Entfernung.  
Distance maximale.

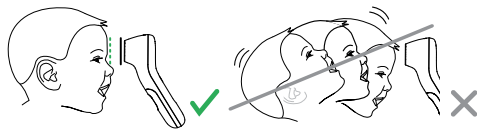


Halten Sie den Kopf ruhig.  
Maintenez la tête immobile.

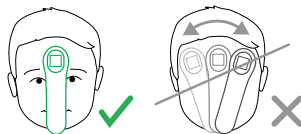


Halten Sie das Thermometer in der richtigen Position.  
Maintenez le thermomètre dans la bonne position.



# 2 Hinweise zur Temperaturmessung Conseils pour la prise de température

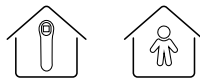

Vor der Messung: Das Thermometer sollte 10 Minuten  
bei Raumtemperatur aufbewahrt werden, die Person  
sollte sich 30 Minuten in Innenräumen aufhalten.  
Avant la mesure : maintenez le thermomètre à  
température ambiante pendant 10 minutes et la  
personne à l'intérieur pendant 30 minutes.

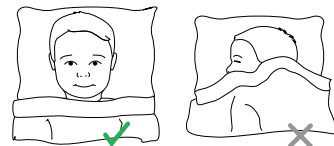

Helen  
of Troy

© 2025, All rights reserved.

Kaz Europe Sàrl  
A Helen of Troy Company  
Q-Center, Chaux 4  
CH-1030 Bussigny - Switzerland

Certain trademarks used under  
license from The Procter &  
Gamble Company or its affiliates.

Sensian™ is a trademark of  
Helen of Troy Limited.

[www.BraunHealthcare.com](http://www.BraunHealthcare.com)

Made in China.

MD

REF BNT050WE

CE 0297



P/N: A006942R2  
30JUL24