BRAUN

ThermoScan® 4

Ear thermometer

IRT3515

GB	English	3
CZ	Česky	15
FI	Suomi	27
PL	Polski	39
RO	Română	51
SA	اللغة العربية	74
DK/NO/SE	Scandinavisk	75
SK	Slovenčina	93
TR	Türkçe	105

Certain trademarks used under license from the Procter & Gamble Company or its affiliates.

ThermoScan® is a registered trademark of Kaz Europe Sàrl and/or its affiliates.

**Helen
of Troy**

 Kaz Europe Sàrl
A Helen of Troy Company
Q-Center, Chaux 4
CH-1030 Bussigny - Switzerland

 0297

Printed in China

© 2025, All rights reserved.

www.HelenofTroy.com/emea-en/

www.BraunHealthcare.com

Made in China


MD **REF** IRT3515EE

P/N: A009414R1
01JUL25

Product description

- ① Protective cap
- ② Hygiene Cap
- ③ Probe
- ④ Start button
- ⑤ Probe tip (lens)
- ⑥ Hygiene Cap detector
- ⑦ Display
- ⑧ Battery door safety lock
- ⑨ Battery door (1 X 3V type 2032 battery)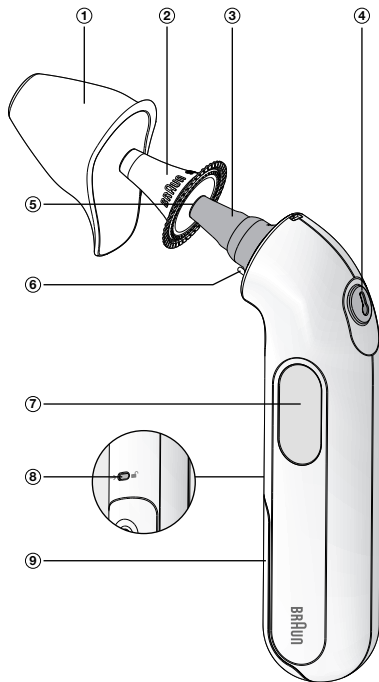

Intended use and contraindications

The Braun ThermoScan® 4 Ear thermometer is intended to be used by home users for intermittent measurement of the body temperature of patients having ages ranging from normal weight (full term) new-born to geriatric adult. The thermometer is reusable.

The Braun Hygiene Cap is a single use device and is intended to maintain accuracy of measurement, hygiene and help prevent cross-contamination.

This thermometer is not intended for clinical use in a professional environment and is for home usage only. This thermometer is not intended to interpret hypothermic temperatures.

Do not allow children under 12 to take their temperatures unattended.

This product is not intended to diagnose any disease, but is a useful screening tool for temperature. Use of this thermometer is not intended as a substitute for consultation with your physician.

Warnings and Precautions

Please consult your physician if you see symptoms such as unexplained irritability, vomiting, diarrhea, dehydration, changes in appetite or activity, seizure, muscle pain, shivering, stiff neck, pain when urinating, etc. in spite of absence of fever.

Please consult your physician if the thermometer shows elevated temperature (yellow or red background lights).

The operating ambient temperature range for this thermometer is 10 – 40 °C (50 – 104 °F).

Do not expose the thermometer to temperature extremes (below –25 °C / –13 °F or over 55 °C / 131 °F) or excessive humidity (> 95 % RH).

This thermometer must only be used with genuine Braun ThermoScan® Hygiene Caps. Other probe covers can lead to inaccuracy.

To avoid inaccurate measurements always use this thermometer with a new, clean, Hygiene Cap.

If the thermometer is accidentally used without a Hygiene Cap, clean the lens after use (see «Care and cleaning»).

Keep Hygiene Caps out of reach of children.

This thermometer must stay in stable ambient (room) temperature for 30 minutes before operating.

Temperature elevation may signal a serious illness, especially in adults who are old, frail, have a weakened immune system, or neonates and infants. Please seek professional advice immediately when there is a temperature elevation and if you are taking temperature on:

- neonates and infants under 3 months (consult your doctor immediately if the temperature exceeds 37.4 °C or 99.4 °F)
- Individuals over 60 years of age
- Individuals having diabetes mellitus or a weakened immune system (e.g., HIV positive, cancer chemotherapy, chronic steroid treatment, splenectomy)
- Individuals who are bedridden (e.g., nursing home patient,

stroke, chronic illness, recovering from surgery)

- a transplant recipient (e.g., liver, heart, lung, kidney)
- Do not allow children to take their temperatures unattended.

People who are on antibiotics, analgesics, or antipyretics should not be assessed solely on temperature readings to determine the severity of their illness.

Do not modify this equipment without the authorization of the manufacturer.

This thermometer contains small parts that can be swallowed or produce a choking hazard to children. Always keep the thermometer out of children's reach.

How to use your Braun ThermoScan® 4

Before taking the temperature, make sure the ear is free from obstructions or excess earwax.

Do not attempt to take the reading without a Hygiene Cap on – the measuring sensor must be kept protected and clean from earwax to give accurate readings.

Change the Hygiene Cap with each use to ensure hygiene, to help prevent cross-contamination and to ensure earwax residues on the Hygiene Cap do not impact accuracy of the reading.

1. Power on

To turn the thermometer on, push the «start» button  (4).

During an internal self-check, the display shows all segments.

The reading shown on the screen is the last temperature taken.

Flashing symbol of Hygiene Cap will appear if no Hygiene Cap is attached. Temperature can be taken only if Hygiene Cap is mounted on the probe.


The thermometer is ready to use after two beeps are sounded and the non-flashing ear icon appears.

2. Take temperature

In clinical studies, precision and accuracy of ear thermometer measurement has been shown to be influenced by variability in user technique. Fully insert the thermometer probe into the ear canal and point it towards the ear canal, make sure the thermometer is pointed toward the ear drum and not the canal walls. Once positioned correctly, press and release the start button (4).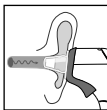

Keep the thermometer steady in the ear canal.

Temperature in right and left ear differs, always compare measurements from the same ear. The natural physiological difference may reach 0.2 °C. In the absence of confounding factors, in high-risk population (like infants or immunodepressed people) use the highest of the two to interpret the patient's condition.

External factors may influence ear temperatures, including when an individual has:

- been lying on one ear or the other
- had their ears covered
- been exposed to very hot or very cold temperatures, or
- been recently swimming or bathing

In the situations above, wait 20 minutes prior to taking a temperature.

If ear drops or other ear medications have been placed in the ear canal of one ear, take the temperature in the untreated ear.

One beep signals normal temperature. Audio fever guidance will trigger 2 consecutive beeps when the reading is equal or higher to 37.5 °C (99.5 °F) and 4 beeps if the reading is equal or higher to 38.6 °C (101.5 °F). The result is shown on the display (7).

Colour-coded fever guidance


Colour-coded fever guidance helps you to better understand the meaning of the temperature with the color indicated on the display. The screen displays green for a normal temperature, yellow for a fever, and red for a high fever.

Color range	Reading	Meaning
Green	≥ 35.8 – ≤ 37.4 °C (≥ 96.4 – ≤ 99.4 °F)	Normal
Yellow	≥ 37.5 – ≤ 38.5 °C (> 99.4 – ≤ 101.3 °F)	Fever
Red	> 38.5 °C (> 101.3 °F)	High fever

3. Remove Hygiene Cap

Gently pull off the Hygiene Cap (2).

Do not use force in removing the Hygiene Cap.

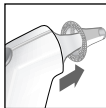4. Use a new Hygiene Cap

Put on a new, clean, Hygiene Cap by snapping into position.

To ensure an accurate reading:

Use a clean Hygiene Cap with each measurement:

- To ensure hygienic use
- To help protect from cross-contamination
- To ensure earwax residues do not impact accuracy of the reading.


Earwax and dirt on the tip of the Hygiene Cap, even in microscopic layers, can create an obstacle between the sensor and the infrared heat emitted by the ear and impact the accuracy of the measurement.

5. Turn off

Place the protective cap on (1).

The Braun ThermoScan® 4 Ear thermometer turns off automatically after 60 seconds of inactivity. The thermometer can be turned off by pressing and holding the start button for more than 5 seconds.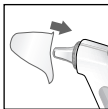


Care and cleaning

The probe tip (5) is the most delicate part of the thermometer. It has to be clean and intact to ensure accurate readings.

If the thermometer is accidentally used without a Hygiene Cap, clean the probe tip as follows:

Very gently wipe the surface with a cotton swab or soft cloth moistened with alcohol. Do not use cleaners other than alcohol on the probe tip. Use of other cleaners could damage the tip.

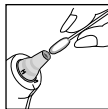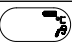
Once the alcohol has dried completely, a new Hygiene Cap can be put on and a temperature measurement taken. If the probe tip is damaged, please contact your authorized local service center.

Use a soft, dry cloth to clean the thermometer display and product exterior. Do not use abrasive cleaners. Never submerge this thermometer in water or any other liquid.

Store the thermometer and Hygiene Caps in a dry location free from dust and contamination and away from direct sunlight.

Additional Hygiene Caps (LF40) are available at most stores selling Braun ThermoScan®.

Trouble-shooting

Situation	Solution
 No Hygiene Cap is attached.	Attach new, clean, Hygiene Cap.
 Measurement before device stabilization.	Wait until all the icons stop flashing.
 Ambient temperature is not within the allowed operating range (10–40 °C or 50–104 °F).	Allow the thermometer to remain for 30 minutes in a room where the temperature is between 10 and 40 °C or 50 and 104 °F.
 Temperature taken is not within typical human temperature range (34–42.2 °C or 93.2–108 °F).	Make sure the probe tip and lens are clean and a new, clean, Hygiene Cap is attached. Make sure the thermometer is properly inserted. Then, take a new temperature.
 System error.	Reset the thermometer by removing the battery and then replacing it again, wait 1 minute, then turn on again. Please contact your authorized Service Center.
 Battery is low.	Insert new battery.
 Battery is too low. Device cannot be powered on to the ready state.	Insert new battery.

How to change the temperature between °C and °F

Make sure the thermometer is turned off. Press and hold down the <<start>> button. Keep holding down the <<start>> button and after about 3 seconds the display will start to cycle between temperature scales (°C or °F).



Release the start button when the desired scale is shown. This will reset the thermometer to the new temperature scale.

Replacing the battery

The thermometer is supplied with one 3V Lithium cell battery (type CR2032). Insert new battery when the low battery symbol appears on the display.

1. The battery door design is intended to prevent a small child from easily opening the battery door to address a possible choke hazard. Open the battery compartment by inserting a narrow pointed object such as a ball point pen or paperclip into the small hole above the battery door (8).
2. While pressing into the hole slowly slide back the battery door with the other hand.
3. To close battery door align the tabs along the side of the battery into the battery compartment and slowly slide back into position until the door clicks into its locked position. If the battery compartment does not close securely, stop using the thermometer and keep it away from children.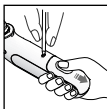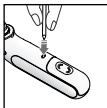

BATTERY WARNING KEEP OUT OF REACH OF CHILDREN. Swallowing can lead to chemical burns, perforation of soft tissue, and death. Severe burns can occur within 2 hours of ingestion. Seek medical attention immediately.

Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your retailer.

Calibration

This product has been calibrated at the time of manufacture. If used according to the use instructions, periodic calibration is not required. This device is not intended as a substitute for regular check-ups by your doctor, please consult your doctor if you have any doubt about the temperature reading. LOT and Serial numbers of your product must be provided as they are essential to record and follow up your inquiry or claim. The LOT and Serial numbers are located in the housing.

Inquiry

In Europe, any serious incident (e.g. death, life-threatening injury, surgical intervention, ...) that has occurred in relation to this thermometer should be reported to Kaz Europe (see Helen of Troy Authorized Service Centers) and to the competent authority of your member state.

The production date is given by the LOT number and can be deciphered as explained below:

The first 3 numeric digits in the LOT number represent the day of the year of manufacture. The next 2 numeric digits represent the last two numbers of the calendar year of manufacture and the letter(s) at the end designate the manufacturer of the product. (E.g.: LOT No.: 12313tav this product was made on the day 123, year 2013 at the manufacturer code tav.)

2 Year Limited guarantee

Please read all instructions before attempting to use this device.

Please retain the receipt as proof of and date of purchase. The receipt must be presented when making any claim within the relevant guarantee period. Any claim under the guarantee will not be valid without a proof of purchase.

Your device is guaranteed for Two Years (2 Years) from date of purchase.

This guarantee covers defects in materials or workmanship that occur under normal use; defective devices meeting these criteria will be replaced free of charge.

The guarantee DOES NOT cover defects or damage resulting from abuse or failure to follow the user instructions. The guarantee becomes void if the device is opened, tampered with, or used with non-Braun branded parts or accessories, or if repairs are undertaken by unauthorized persons.

Accessories and consumables are excluded from any guarantee.

For support requests, please visit www.BraunHealthcare.com/uk_en/ or find service contact information at the end of this owner's manual.

This Guarantee is applicable to Europe, Russia, Middle East, and Africa only.

UK Only: This does not affect your consumer statutory rights

Product specifications

Model:	IRT3515
Displayed temperature range:	34 °C – 42.2 °C (93.2 °F – 108 °F)
Operating ambient temperature range:	10 °C – 40 °C (50 °F – 104 °F)
Operating relative humidity:	15% to 95% non-condensing
Storage relative humidity:	15% to 95% non-condensing
Storage temperature:	-25 °C – 55 °C
Display resolution:	0.1 °C or °F

Accuracy for displayed temperature range:

± 0.2 °C (35.5 – 42 °C)
(95.9 – 107.6 °F)
± 0.3 °C (outside this temperature range)

Clinical bias:

-0.11 °C (-0.198 °F)

Limits of agreement:

± 0.25 °C (0.45 °F)

Clinical repeatability:

± 0.3 °C (0.18 °F)

Measuring Site:

Ear

Reference Site:

Oral (estimated oral temperature is displayed)

Battery Type:

one 3V Lithium cell battery
(Type CR2032)

Battery life:

3 years /1000 measurements

Service life:

5 years

Accessory:

Hygiene Cap, LF40/LF20, single use

If device is not used or stored within specified temperature and humidity ranges the technical accuracy cannot be ensured.

Refer to Instruction manual
(blue symbol)

Equipment with
type BF applied parts

Consult electronic
instructions for use

Operating temperature

Storage temperature

Batch code

Serial number

Catalogue number

Medical device

Manufacturer

Hygiene Cap: Do not re-use

Authorized representative in
the European Community

Unique device identifier

Operating humidity

Storage humidity

Subject to change without notice.

IP22: Protected against solid foreign objects of 12.5 mm diameter and greater. Protected against vertically falling water drops when the device is tilted up to 15 degrees.

This medical device carries the CE mark and is manufactured in conformity with RoHS Directive 2011/65/EU and other applicable directive and/or regulations as specified in the EU Declaration of Conformity. Full responsibility for the conformance of the product to the standard is assumed by Kaz Europe Sarl, Switzerland.

This product needs special precautions regarding EMC. Portable and mobile RF communications equipment can affect medical electrical equipment. **WARNING:** Use of this equipment adjacent to other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally. **WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30cm to any part of the device. Otherwise, degradation of the performance of this equipment could result. **WARNING:** use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation. For more information, see consumer service numbers or website.

Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.

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✉ Braun_Support@helenofroy.eu

LEGEND INFO DO NOT PRINT THIS PAGE

Helen of Troy		Creative Department Artwork Specifications	
Brand:	Braun		
Category:	Thermometer		
Model:	IRT3515EE		
Artwork Part #:	A009414R1		
Die Line Part #:	N/A		
Subject:	Owner's Manual		
Region:	EMEA		
Flat Size:	MM: W156 x H120		
Folded Size:	MM: W78 x H120		
Scale:	1/1		
Material:	50# matte text (inside pages) Minimum 80# gloss or matte text (cover)		
Page Count:	120		
Revision:	1		
Date:	01JUL25		
Release Date:	08JUL25		
Re-release Date:			
Colors:	Dielines (Do not print) Spot Colors		
Cyan	Magenta	Yellow	Black
0%	0%	0%	100%
Special Instructions:			
Quality Requirement of Artwork and Quality Clarification Process of Artwork Printing:			
Meet Eng-GS-06&02			
Helen of Troy Creative Services Marlborough, MA 01752, USA +1 508 490 7000			

LEGEND INFO DO NOT PRINT THIS PAGE