BRAUN

Manual nasal aspirator 1 BNA050US

Contact Us

Questions or Comments?

Call us toll-free at: 1-800-327-7226

E-mail: ConsumerRelations@HelenofTroy.com

www.BraunHealthcare.com

Certain trademarks used under license from The Procter & Gamble Company or its affiliates.

English

Please read all the instructions before using the Braun Manual nasal aspirator, especially this basic safety information.

- The manual nasal aspirator is designed to clean/extract nasal secretions only. **DO NOT** use the device for purposes other than those stated in these instructions for use.
- Keep out of reach of children under 12 years.
- **DO NOT** expose the Braun Manual nasal aspirator to direct sunlight.
- This device is designed for home use only. Consult a doctor if the child is suffering from an upper respiratory infection.
- **DO NOT** use other nozzles than those supplied with this device.
- Only use Braun Single use nasal aspirator filters with this device.
- **DO NOT** reuse filters.

Limited one year warranty

You should first read all Instructions before attempting to use this product.

- A. This 1 year limited warranty applies to repair or replacement of product found to be defective in material or workmanship. This warranty does not apply to damage resulting from commercial, abusive, unreasonable use or supplemental damage. Defects that are the result of normal wear and tear will not be considered manufacturing defects under this warranty. KAZ IS NOT LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES OF ANY NATURE, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS LIMITED IN DURATION TO THE DURATION OF THIS WARRANTY. Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you also may have other rights which vary from jurisdiction to jurisdiction. This warranty applies only to the original purchaser of this product from the original date of purchase.
- B. At its option, Kaz will repair or replace this product if it is found to be defective in material or workmanship.
- C. This warranty does not cover damage resulting from any unauthorized attempts to repair or from any use not in accordance with the instruction manual.

Call us toll-free at 1-800-327-7226, e-mail: ConsumerRelations@HelenofTroy.com, or visit our website at: www.BraunHealthcare.com. Please be sure to specify a model number.

NOTE: IF YOU EXPERIENCE A PROBLEM, PLEASE CONTACT CONSUMER RELATIONS FIRST OR SEE YOUR WARRANTY. DO NOT RETURN THE PRODUCT TO THE ORIGINAL PLACE OF PURCHASE.

Device specifications

Model	BNA050US
Operating temperature	50 – 104 °F
Storage temperature	-13 – 131 °F
Operation & storage relative humidity	≤95 % R.H. non-condensing
Service life	5 years
Warranty	1 year

Consult Instructions for Use

Storage temperature

95%

Single-use filter specifications

Model	BNF040US
Operating temperature	50 – 104 °F
Storage temperature	-13 – 131 °F
Operation & storage relative humidity	≤95 % R.H. non-condensing

Single use

Select nose tip for child based on nostril size

Ensure filter is in place and chamber is securely screwed together

Insert nose tip into child's nostril to create a seal

Use mouthpiece to suck out mucus

After use, dispose of filter

Disassemble and clean in sink or dishwasher. Top rack only

Disassemble and boil for 15 minutes or disinfect tube and other components with rubbing alcohol

Place new filter in filter holder

CHOKING HAZARD - May contain small detachable parts, keep out of reach of children under 3 years

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© 2021, All rights reserved.
Kaz USA, Inc., a Helen of Troy Company